



# Lissamine Green Ophthalmic Strip

## DESCRIPTION

Each strip is impregnated with 1.5 mg of Lissamine Green.

Lissamine Green/Green Glo strips have an overall length of 70 mm. The holding paper is 7 mm in width. The impregnated portion is sandwiched between the holding paper and has an exposed portion of 15 mm by 5 mm. The holding paper offers rigidity to the strip which makes it easy and convenient for use. The end of the tip rounded with a radius of 2.5 mm. The absorbent paper holding the dye is made of Whatman # 1 or its equivalent. The holding paper is white in color and non-shredding.

Each strip is individually packed and sealed in glassine paper to render a sterile product. The glassine paper has a gsm of not less than 40.

## INDICATIONS

For use as a diagnostic agent when superficial corneal or conjunctival change is suspected.

## CONTRAINDICATIONS

Lissamine Green/Green Glo strips are not meant for use with gas permeable lenses or soft contact lenses as this will stain the lenses. It is known to irritate eyes.

## WARNING

- Keep this and all medication out of the reach of children.
- For external use only.
- Discontinue in case of persistent irritation or burning sensation.
- Do not use if the pouch is found ruptured or damaged.
- This product is meant for professional use only.

## STORAGE

- Store at room temperature 15°-30°C (59°-86°F).
- Keep dry, do not store in moist environment.

## DIRECTIONS

1.  Grasp tabs between thumbs & index fingers

2.  Gently pull tabs apart

3.  Remove strip taking care to touch only the white portion of the strip

4. Handle the exposed strip while holding the white portion of the paper.

5. Moisten the impregnated part of the strip with sterile normal saline. (Alternately tear fluid from the lower fornix may be used.) Have the patient look up, stroke the moistened tip along the bulbar conjunctiva. The patient should then blink several times to obtain best result.

Observe under slit lamp for stained cells. The extent of staining is indicative of corneal epithelial cell damage.

## HOW SUPPLIED

In boxes of 100 sterile strips.

## DISCLAIMER OF LIABILITY/RETURNS

The firm is liable only for potential manufacturing defect. Return the damaged container in its original packing identified by the lot number, your references and reason for return.



**Manufactured by:**  
Omni Lens Pvt. Ltd.  
Factory: Plot No. : 17, Ambota, Sector-5,  
Parwanoo, Dist.: Solan - 173 220,  
Himachal Pradesh, INDIA.  
Email: info@omnilens.in

C&B D-U-N-S Number: 86-217-0057

Mfg. Lic. No. SL-MB/2011-71

**Manufactured for & distributed by:**  
HUB Pharmaceuticals LLC  
6617 N Scottsdale Road, Suite 204  
Scottsdale AZ 85250, USA  
www.hubrx.com



FOR EXTERNAL USE ONLY.

STERILE EO Sterilized by EO

L.L. No.: 669-L-B(H)