

TraxIt® Wearable Children's Underarm Thermometer



— Enables continuous temperature readings for up to 48 hours

— Accurate



— Ideal for monitoring children's temperatures

— Easy to read

— Gentle medical tape backing

— Waterproof

— Safe, Latex-free, non-mercury

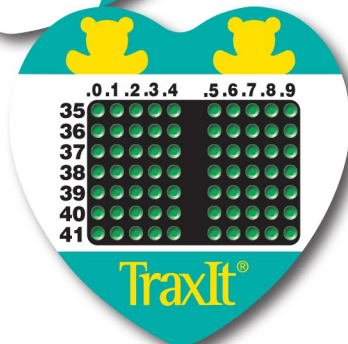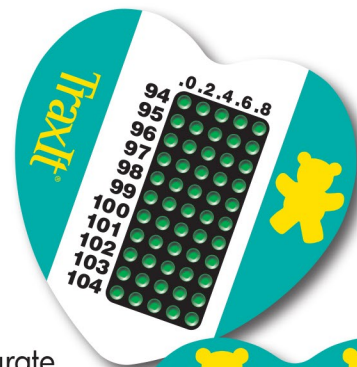

Directions:

1. Peel off thermometer from backing and place on the side of the body in the axilla (armpit).
2. Close arm and wait 2-3 minutes for the first temperature measurement – shorter times may give inaccurate readings.

How To Read:

1. Lift arm to read. Thereafter readings are virtually instant. Just lift the arm to take the next measurement.
2. Look at dot grid. The highest black dot indicates the correct temperature (see example).

Note: TraxIt® will adhere to the armpit through bathing and may be left in place up to 48 hours on normal, healthy, intact skin.

To Remove:

Gently peel off and discard. DO NOT REUSE.

Accuracy:


- ±.2°F within the critical range of 98.0°F to 102.0°F;
- ±0.4°F elsewhere.
- ±.1°C within the critical range of 37.0°C to 39.0°C;
- ±0.2°C elsewhere

No Temperature Offset:


TraxIt® displays the actual axillary temperature. Axillary temperature readings are typically 0.5°F (0.3°C) lower than oral and 1.0°F (0.6°C) lower than rectal.



Ordering Information		
Fahrenheit	Celsius	Quantity
1313-05	2313-05	5 boxes of 100 units
1313-15	2313-15	10 boxes of 100 units
1313-20	2313-20	20 boxes or 100 units

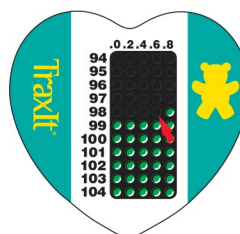

Example: Last black dot reads 98.6°F

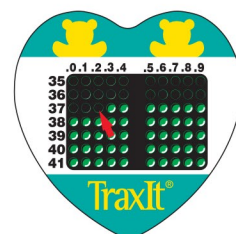

Example: Last black dot reads 37.2°C

FDA Listed
CE Certified
ISO 13485: 2003

 **MEDICAL INDICATORS**

16 Thomas J Rhodes Industrial Drive, Hamilton, NJ 08619
Phone: 609-737-1600 Fax: 609-587-8635
info@medicalindicators.com web: www.medicalindicators.com